# LUXEN Zirconia CAD/CAM Block (LUXEN Zr, LUXEN Smile)

INSTRUCTION FOR USE (DM-DMF-100 VER. 4.1 2019.10.17)

## Indication for use

LUXEN Zr: Full crown, Up to 6 units with no more than 2 pontics between the pillars, Inlays or onlays, Veneers

LUXEN Smile: Full crown, 3-unit bridges, Inlays or onlays, Veneers

## Type, Class(ISO 6872:2015)

LUXEN Zr(Type 2, Class 5)

LUXEN Smile(Type 2, Class 4b)

## Composition

LUXEN Zr:  $ZrO_2+HfO_2+Y_2O_3$  : >99.8wt. %( $Y_2O_3$ :4-6wt. %)

LUXEN Smile:  $ZrO_2+HfO_2+Y_2O_3$  : >99.8 wt. %( $Y_2O_3$ : 7-10wt. %)

## Colors:LUXEN Zr (Smile) colored shade use for VITA Shade guide;

White— A0, S1- A1, S2- A2, S3- A3, S4- A4, E0- B1, E1- B2, E2- B3, E3- B4, E4- C4

Multi-white- A1-A2-B3-B4 LAYER, Multi-ivory- B1-B2-B3-B4 LAYER and Multi-reddish- A1-A2-A3-A4 LAYER.

## Size(mm) (Width: W, Length: L, Height:H, Diameter:D)

Block type: 16/20/40/55/65/72 (W), 19/23/24/25/35/40(L), 10/12/14/15/16/18/20/22/25(H)

Disk type: 98/98.3/100(D), 10/12/14/16/18/20/22/25(H)

Wieland type: 98.3/100 (D) 10/12/14/16/18/20/22/25 (H)

D95 type: 95(D) 10/12/14/16/18/20/22/25/30 (H)

## Sintering ScheduleChart of Sintering Parameters–Final Temperature

Step	Temperature	Heating rate	Holding time
1 <sup>st</sup>	Room Temp - 1000°C	max 9°C	10min
2 <sup>nd</sup>	1000°C - 1500°C	max 3.5°C	2hr
3 <sup>rd</sup>	1500°C – Room Temp	max 9°C	

Product	Final temperature
LUXEN Zr/LUXEN Smile (Pre shade&Mutli)	1500°C
LUXEN Zr ST	1580°C
LUXEN Smile(White)	1450°C

## Technical Data

Product	Unit	Coefficient of thermal expansion CTE(20-500°C)
LUXEN Zr	$10^{-6}/K^{-1}$	$10.5 \pm 0.5$
LUXEN Smile	$10^{-7}/K^{-1}$	$10.9 \pm 0.5$

## Directions for Use

- 1) Take out the disc from the package and confirm that the blank has no crack
- 2) Attachment the disc to the milling machine and start on milling
- 3) After milling, remove the frameworks(full crowns) with use the diamond bur etc
- 4) Finish with tungsten carbide bur etc
- 5) Remove the de-dust with air gun
- 6) Color frame(full crown) with color liquid
- 7) Dry for 40 minutes(above the 4unit bridge is dry for 1hr)
- 8) Put the frameworks(full crown) in the sagger tray and start the sintering furnace
- 9) Remove sintered frameworks(full crown) from furnace at room temperature
- 10) Adjust the frame on working model for trim use the water cooling air turbine with diamond bur, if needed
- 11) Confirm the frameworks(full crown) has no crack
- 12) After polishing use the build-up and glaze

## Warning

- 1) The product must not be used on following patients.
  - Patient with allergic reaction with components of product
  - Patient with bruxism or other abnormal functional activities (patient diagnosed with excessive masticatory function, and patient especially with bruxism and clenching)
- 2) Precautions related to age, gender and health status of intended users with consideration to characteristics of medical devices.
  - Product must not be handled by children or elderly.
- 3) Precautions related to adverse reaction following the use of medical devices and side-effects/accidents following careless handling during the use
  - There is very low probability of side-effects from the product when the product is used appropriately. However, this does not theoretically exclude immune system reaction (eg. allergic reaction) or abnormal function of local perception (eg. provocative taste or oral mucosa irritation).
  - In case of patient with hypersensitive reaction against the product, product must not be used or only used following detail diagnosis by dentist.

#### 4) General Precautions

- Product must be used by medical specialist or dentist with full compliance with user instruction and precautions.
- Product must not be used for the purpose other than medical purposes
- Product of left over following the use must be kept in room temperature avoiding the direct sunlight.
- Single-use medical device must not be re-used.

#### 5) Interactions

- Product must not be used with metal porcelain or alumina porcelain
- Hydrofluoric acid must not be used when pressed ceramics are removed from framework. The acid will dissolve zirconia and result in weakening of product strength.
- Product must not be contacted or immersed in organic solution.

#### 6) Precautions on Application

Application must be avoided in following situations.

- Insufficient wall thickness
- Direct insertion without polishing or glazing
- Cutting block that used incompatible CAD/CAM system
- Use of stain or veneering material from other manufacturer
- Accessory use of firing paste from other manufacturer
- Use of tungsten carbide cutter

#### 7) Precautions Related to Prevention of Safety Hazards

- Avoid contact with skin and eye. In case of contact, rinse thoroughly with water and consult with doctor.
- Store product at a place that is out of reach from young children.
- Care must be given with damage during transport or storage in relation to shock, pressure, vibration, distortion, and shear force.

#### Storage and Handling

- 1) Do not give a shock
- 2) Keep away from direct sunlight
- 3) Keep it at normal temperatures: 1°C-30°C

 **DENTALMAX Co., Ltd**  
50-7, Pungsesandan 2-ro, Pungse-myeon, Dongnam-gu, Cheonan-si,  
Chungcheongnam-do, 31217, Korea

TEL)+82-41-523-3580, FAX)+82-41-523-3581

 **BIOGERI**

Schanzenstrasse 10, Hamburg, 20357, Germany  
Tel: +49 40 4143 10 215 0  
Fax: +49-40 4143 10 215 9

Manufacture

Authorized representative  
in the European  
Community

Batch code

Date of manufacture

Non-sterile

Do not reuse

Catalogue number

CE (Notified Body)

Consult instruction for use

Do not use if package is  
damaged

Temperature limitation

**Rx only**

**CAUTION:** Federal  
law(USA) restricts sale of  
this device to or on the  
order of a physician,  
dentist, or licensed  
practitioner.

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